UPLOADGUIDE

NEW CareLink™ Personal uploader

Your first step to understanding more about your diabetes.



BEFORE YOU BEGIN, GET EVERYTHING READY

Plug your USB or meter into your computer

- CareLink USB or meter plugged in and placed as close as possible to the pump
- Keep the meter linked with the pump



Make sure your pump is ready to upload

Check that the pump is not on one of these screens.







- ☐ No temp basals
- ☐ No alarms
- ☐ No active bolus
- \square No low battery

Medtronic

INITIAL FOUR-STEP PROCESS TO ADD A NEW PUMP **OR METER**



Choose pump or meter

Pump



Select the pump model



Enter pump serial number



Make sure the pump is ready

Click "Add Pump"

Meter



2 Select the meter brand if you want to upload BG meter only data



Select meter model



Make sure the meter is ready

Click "Add Meter"

TWO EASY STEPS TO UPLOAD PUMP

OR METER DATA



Select the device



- Make sure the device is ready to upload
- Click "Upload Now"



Link to your doctor's CareLink system for more efficient data sharing.

UPLOAD GUIDE

CareLink™ Personal uploader

Safety Information: CareLink™ software

CareLink software is intended for use as a adjunctive tool to help manage diabetes and is NOT intended to be relied upon by itself. For more details, please consult the User Guide at medtronic-diabetes.in/ support/guides-and-manuals.

© 2018 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company. 950M17620-011 201701001

India Medtronic Pvt. Ltd. Solitaire Corporate Park, Bldg. No 12, 1241, 4th Floor, Andheri-Ghatkopar Link Road, Andheri (E), Mumbai 400093 +912248810700/701

UC201807716 EN